

Kobayashi Preferred DSAEK Set

Akira Kobayashi, MD, PhD, Kanazawa, Japan




Kobayashi DSAEK Marker: AE-2868

"While most DSAEK Markers mark in a complete circle which causes the ink to run, the Kobayashi DSAEK marker marks in dots along the circumference. This prevents the ink from running so that visibility is good through out the procedure."




Kobayashi DSAEK Chamber Maintainer (25G): AE-7802

"The Kobayashi Chamber Maintainer was designed for DSAEK but can be used for various procedures in maintaining the AC. The surface of the cannula has been sandblasted so that it stays in place. I recommend using a 24G MVR blade to create the incision."

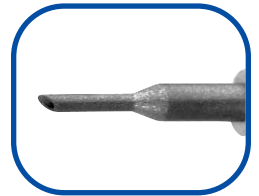

Tan DSAEK Forceps: AE-4226

"During DSAEK pull-through technique, I had some problems with the use of capsulorhexis forceps, since its curve and tip design is not optimized for DSAEK. However, the use of Tan forceps, specifically designed for DSAEK, enables quite smooth insertion of the donor lamella into the anterior chamber. The curve and tip of the forceps is well designed, and suited for pull-through technique of DSAEK."

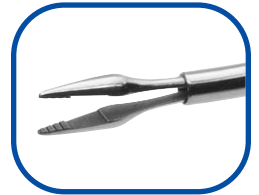

Kobayashi DSAEK Donor Adjuster (30G): AE-7804

"Usually after inserting the donor button into the AC, a Sinskey hook is inserted to position the button in place and then an air injection cannula is inserted to release the air bubble into the AC. This instrument exchange tends to cause the donor button to shift so that it does not adhere properly. The surgeon is forced to readjust the button in place. In order to simply this process, the Kobayashi DSAEK Donor Adjuster acts as both a hook and cannula. Now the surgeon can simultaneously hold the button in place while injecting the air bubble. With this new cannula, successful adherence can be achieved safely and easily."



The more you know...
ASICO
Today's Precision... Tomorrow's Vision™

ASICO LLC
26 Plaza Drive • Westmont, IL 60559 • USA
Phone: 630-986-8032 • Fax: 630-986-0065
Customer Service: 1-800-628-2879
www.asico.com • E-mail: info@asico.com



© 2009 ASICO LLC

BR310-09